# BureDouble 2.0

System**RoMedic**

---

Instructions for use – English

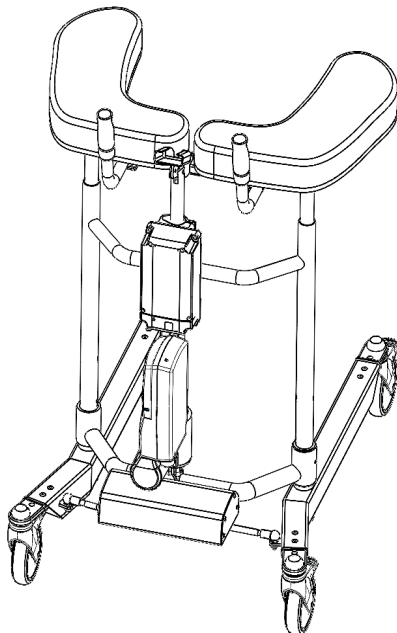
Bruksanvisning – Svenska

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---

SWL: 150 kg / 330 lbs

---

## Table of contents

English .....	3
Svenska .....	18
Danske .....	32
Deutsch .....	46
Suomi .....	60
Français .....	74

# Product Overview

**REF**

56-303  
56-303-AUS  
56-303-B/0-FKHW\_R  
56-303-B/0-FKHW\_R-PU  
56-303-B/0-PU  
56-303-B/1-PU  
56-303/75  
56-303/75-B/0  
56-303/75-B/0-FKNW\_R-PLUS  
56-303/75-B/0-PU  
56-303/75-UK  
56-303/75-TKN\_R  
56-303/100  
56-303/100-PU  
56-303-PLUS  
56-303-PLUS-PU  
56-303-PU  
56-303-PU-AUS  
56-303-UK  
56-303-TKN\_R

The Bure Double 2.0 is equipped with twin electric drives for raising and lowering the Walker and widening the walking frame. The new version has a significantly lower height above the walking frame for better access and a higher user weight (150 kg) compared to the earlier version.

### 1. Intended use

Walkers are to be used as indoor, movable and portable assistive aids, which together with approved accessories are intended to be used for transfer from sitting to standing positions, and walking training. Walkers are to be used by persons with reduced muscular strength, who shall be able to stand and perform simple leg movements. Walkers are to be used for leaning on when walking and standing during early mobilisation training. Walkers and accessories are intended for indoor use only in Home Healthcare environment and Professional Healthcare facility environment.

### 2. Contraindications, Precautions and Warnings

<b>Contradictions</b>
The device may not be used by patients above the maximum weight indicated on the label of the device.
<b>Precautions</b>
Check that the device is used on flat surface.
Check the device is correctly mounted/assembled before its first use.
Check the device after every folding after any transport.
Check height adjustment and leg spreading motions and inspect the actuators' or gas pistons' full ranges.
Activate the brakes whenever the device is not in use.
Check the function of the brakes before use.
Inspect the device regularly to detect any signs of damage. If there are any visible signs of damage, contact Direct Healthcare Group Customer Service.
In case of an electrical model of the device, check that the hand control does not show signs of wear before every use.
Check that the hand control markings are in accordance with the lifting functions.
Check the battery status of an electrically operated model, before using the device.
It is important to never leave the user alone during the transfer.
Warranty applies only if repairs or alterations are made by personnel who are authorized by Direct Healthcare Group.
Ensure that there are no obstacles or people in the way of the device, when moving.
Handle batteries with care. Do not drop.
Use only batteries and cables that are intended for the device as instructed by the manufacturer.
Verify that potential accessories that might be used are properly attached to the device before use.
Low speed is recommended when moving with the device.
Take care not to drive the device over thresholds with high speed or force. Approach the threshold with lower speed and communicate the coming threshold to the user.
Keep the instructions for use together with the product so that it can be used as a reference if necessary.
Beware of moving parts during usage.
Perform a risk assessment on the user before using the Walker to ensure they can stand and perform simple movements.

<b>Warnings</b>
The caregiver must be able to read and understand the Manual/IFU of the product.
It is important to use only approved accessories to prevent unintended detachment of components and subsequently a fall that may lead to patient injury.
Use careful and gentle maneuvers when moving the device.
Perform maintenance/service of the device, according to the instructions in the manual/IFU, at least once every 12 months.
Accessories must be properly fitted and tested in relation to the user's needs and functional ability.
Special care must be taken when using strong electrical power sources such as diathermy and the like so that diathermy cables are not placed on or near the device. In case of doubt consult with a Direct Healthcare Group representative.
When lifting the device, only use the denoted handholds. At least two people are required for lifting the device.
Do not leave a user or patient unattended when the device is being used.
Never move the lift by pulling on the actuator or gas piston.
The device must not be lowered into water.
The device must not be cleaned using steam.
The device must not be left or stored in a damp or humid environment.
The device must not be charged in a wet room.
The device must not be used outdoors, only indoors.
To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.
No modification of this equipment is allowed
Do not modify this equipment without authorisation of the manufacturer
If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of this equipment.

**Before use:**

- Inspect the packaging for any damage.
- Check for correct product being delivered.
- Check to ensure that the device is free from damage.
- Check if all the parts/components are included in the packaging.
- Check if all approved accessories are included in the package.
- Check if Product Quality Approval document is included in the packaging. Save this document for future contact with manufacturer.
- Make certain the device is properly assembled.
- Check lifting function and bottom-frame adjustment function.
- Check if it is possible to move the device in all directions.

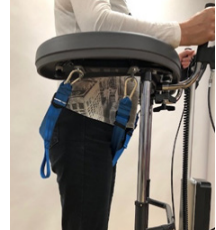
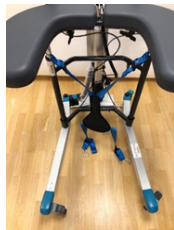
### 3. Standing Up Using the Bure Double

1. Ask the user to sit on the edge of the bed or chair. Bring out the walking table and place it in front of the user. Lock the brakes on the wheels.
2. Ask the user to place their arms on the armrests. If necessary, ask the user to move forward on the seat base so that the user reaches the handles.
3. Ask the user to lean forward. To start raising the walking table, press the UP button on the controller. Raise the walking table to a desired height.
4. Unlock the brakes. Assist the user while walking.

### 4. Using Bure Walker with Gait training Kit 56-388

- The gait support harness has four blue straps with a plastic buckle allowing for adjustments in length, having steel carabiner clips at the ends to be fasten to the gait walker. The carabiner clips on to the metal mounts placed under the table of the Gait walker.
- Apply the gait walker support harness by clipping on the two front carabiners to the front mounters, one on each side of the table.
- Place the user standing up in the Gait walker. Pull back the harness in between the legs and clip on the carabiners to the back mounters, one on each side of the table. Adjust the harness to the user by pulling in the back strap loops.
- **Observe!** Intended use of the Gait trainer support harness is to practise walking. It is **not** a sitting or lifting aid.

## 5. Raising/Lowering the Frame

The desired height is adjusted with Arrow <UP> or Arrow <DOWN>.  
Adjust bottom frame using <WIDER> arrow or <NARROWER> arrow.

Handset from Plus also have charge indicator / battery (low, medium, high) and Service indicator (see further under "16 Hand Control Plus").

## 6. Adjusting the Handles

To adjust handles to the desired position; loosen screw, adjust handle, tighten screw.

## 7. Left and Right Adjustment of Armrests

The handles are adjusted to the desired position by:  
Loosening the knob of the handles adjustable locks.

## 8. Locking Castors/Unlocking Castors

Bure Double is fitted with braked casters with directional locking (green pedal). Depress the foot brake to set it and lock the castor.  
To unlock, press down on the protruding part of the brake. Locking affects rolling and swiveling.

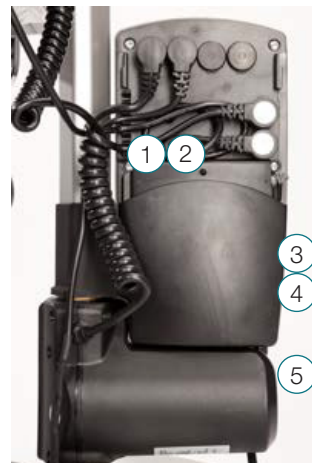
## 9. Control Box Wiring

Remove the control box cover to make the connections as described below. Use a screwdriver or similar; refer to the special labels on the control box.

NOTE: Make sure that plugs are connected to the correct sockets otherwise the actuators can be damaged.

1. Socket for the height adjustment actuator.
2. Socket for the width adjustment actuator.
3. Hand control socket.
4. Battery terminal socket.
5. Plug socket (NOTE: Plugs must ALWAYS be connected to the control box for IP classification to be valid).

## 10. Battery Charging

- Plug needs to connect to 230V wall socket. Charging starts automatically.
  - LED / battery flashes green.
- The battery is of the LiOn type and insensitive to how long / how often you charge the battery. Full charge on discharged battery takes about 24 hours.
- We recommend that the battery plug be removed from the control box if the product will be out of use for an extended period (more than one week) or if the battery is switched off by means of the ON/OFF switch. This eliminates the risk of disabling the battery as the control box always uses a small quantity of electricity even when the Walker is not in use.

## 11. LEDs on Control Box and Battery

Control box – green light when power is available from battery/mains socket.

Battery:

- The LED flashes green during charging.
- ON/OFF: To use the product, press ON.
- The green lamp will begin to flash.
- Use the OFF button (hold down for 5 s) to switch off the battery if the product will not be used for an extended period.
- The LED will show a constant green light when the battery is fully charged.
- When the battery is in normal state/in operation, the LED flashes green at REGULAR intervals, namely 0.5 s ON, 4 s OFF.
- When the battery requires charging (remaining charge <25%) the LED will flash orange/red at REGULAR intervals, namely 0.5 s ON, 4 s OFF.

NOTE: The battery has an in-built device that automatically switches it off when remaining charge is <20%. The LED is then extinguished. The battery must now be fully charged before the electronics can be used again.

## 12. Maintenance

Check that everything is clean and working before using.

Recommended at least once per year

Check that	Action
<p><b>General</b>            The Walker feels firm/does not feel loose.            The Walker does not rattle during manoeuvres.            The Walker is level and all of its castors are in contact with the floor.            The Walker is not dirty.</p>	<p>Clean with lukewarm soap solution or alcohol-based cleaning agent (no petroleum products).</p>
<p><b>Arm rests</b>            Armrests are intact and clean.            Armrest widening is functional.</p>	<p>Clean; replace if damaged.            Replace lock knob.</p>
<p><b>Handle</b>            Grips are not damaged/dirty.            Handle adjustment is functional.</p>	<p>Clean with lukewarm soap solution (no petroleum products).            Replace handle grips.            Fit lock screws/plates as required.</p>
<p><b>Battery</b>            Check that the charging cable is connected to the control box.            Check that the hand control is connected to the control box.            Connect or fit new hand control as required.            Check that the battery, actuators and control box are not loose.</p>	<p>Fit new charging cable; this must always be connected to the control box (IP65 protection).            Connect or fit new hand control as required.            Refer to the wiring diagram in the control box wiring section.            Tighten; replace with new fasteners as required.</p>
<p><b>Height adjustment</b>            Raising and lowering are functional.            The Walker feels stable at maximum settings.            There is no play between the vertical frame and bottom frame.            Height adjustment clamps lock.            The electric actuator must not feel loose.            Height adjustment via hand control is functional.            The electric actuator must run smoothly at a constant speed.</p>	<p>Fit new guide sleeves (in frame) or end plugs (in chrome tubes) as required.            The part list shows the design and which fasteners must be checked. Contact the DHG if the parts cannot be tightened.            Replace clamps.            Tighten the attachment fitting concerned; replace bolts/lock nuts as necessary.            Check that the battery/hand control/ actuator are connected in accordance as in hte control box wiring section.            Charge the battery. Contact customer services.</p>
<p><b>Frame parts</b>            There is no mechanical damage.            There are no scratches.            End plugs/lower frame fitted.</p>	<p>If damage is present, contact DHG customer services.            Touch up as necessary.            Fit new end plugs.</p>
<p><b>Castors/brakes</b>            Castors roll easily/tread not damaged.            The castors are firmly fastened to the lower frame.            Castor brakes functional on all castors.            Handbrake functional.</p>	<p>Clean or replace castors. NOTE: Castors are always fixed to the lower frame with thread locker or lock nuts.            The castors are sealed and we do not recommend their dismantling; instead, replace the whole castor assembly).            Tighten the castor bolt and fit locking nut or use thread locker (depends on model).            Replace castor assembly.            Adjust the brake or fit new castors.</p>

DHG can provide upon request necessary technical information in order to assist maintenance and repair of the device.

### 13. General Maintenance Advice & Cleaning of Walker

- Clean the walker armrests and handles after every use. Wipe using a mild soap solution/surface disinfectant. However, the product is not corrosion proof.
- With environments with high risk of infections use the walker with PU-cushions (Accessory number 57-361-L and 57-361-R)
- Replace damaged parts immediately; must be done by experienced personnel.
- The product is only intended for indoor use.

### 14. General Maintenance for gait training Sling:

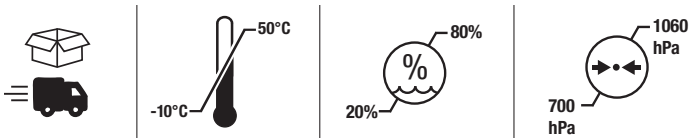
Read the product label.

Do not use rinsing agent.  
To prolong product life,  
avoid tumble-drying.

### 15. Storage and transportation

If the walker is not to be used for some time or e.g., during transport, we recommend that the emergency stop button be pressed in. The walker should be transported and stored in -10 ° C to + 50 ° C and in normal humidity, 20% to 80% non-condensing. The air pressure should be between 700 and 1060 hPa. Leftmost symbol indicates storage and transportation. Let the walker reach room temperature before the batteries are charged or the walker is used. The walker should not be stored so that it is exposed to dust, or so that the battery or gas piston is exposed to direct sunlight.

### Operation

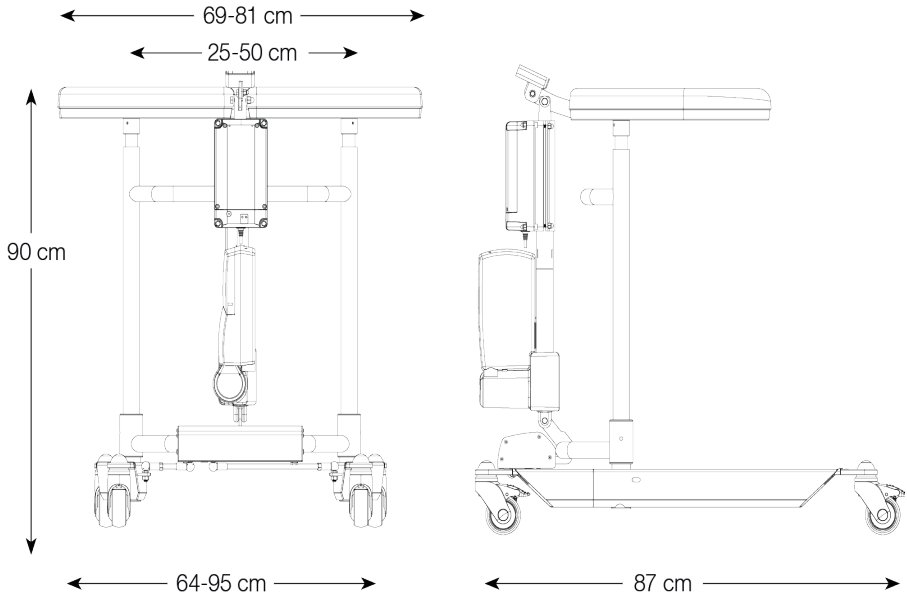
The operating environment should be 5 °C to 40°C, relative humidity 20% to 80% non-condensing, and atmospheric pressure 700 to 1060 hPa. See also Technical Information below. Leftmost symbol indicates operating condition.

## 16. Technical Specifications

Dimensions apply to Walkers with 125 mm castors.

Product Nr.	Width	Max user weight	Length	Height	Weight	Armrest/Internal dimension
56-303	64-95cm	150kg	87cm	90-130cm	40kg	25-50cm

## 17. Recycling/Disposal

When the product is no longer usable it should be recycled in accordance with legislation and regulations in the country concerned. All electrical parts including batteries must be removed and recycled as electrical components. (Ask your local recycling station for further information about how the different materials in the product (metals, plastics, electronics) should be recycled).

## 18. Warranty

For further information or enquiries please contact:

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
info.se@directhealthcaregroup.com  
www.directhealthcaregroup.com

## 19. Electrical System - Technical Data

Actuator	Lift	Widening
Producer	TIMOTION	TIMOTION
Voltage	12V	24V
Push	2000N/1500N	1000N
Pull	2000N/1500N	1000N
Self-locking push	1000N	1000N
Current	6.6 (+/-1,0)A	2.6 (+/-0,4)A
Colour	Black	Aluminium
Stroke	400mm	55mm
Mounting dim	605mm	167mm
Encapsulation	IP66	UIP

IEC60601-1 3rd revision

## Control box + Battery

Producer	TIMOTION
Voltage	24V
Capacity	2.0AH
Encapsulation	IP66
Colour	Black
Electrical safety	Class II

IEC60601-1 3rd revision

## 20. Hand Control Plus

\*\*1. Service indicator – flashes red after 11 months' operation.

Constant light (red) after 12 months' operation.

The indicator is a reminder to review the maintenance instruction (see Maintenance Instructions). This should be done at least once every 12 months.

Reset: hold down buttons <2a> + <2b> for at least 30 seconds.

\*\*The indicator will go out around 30 seconds after the hand control is activated (battery conservation function).

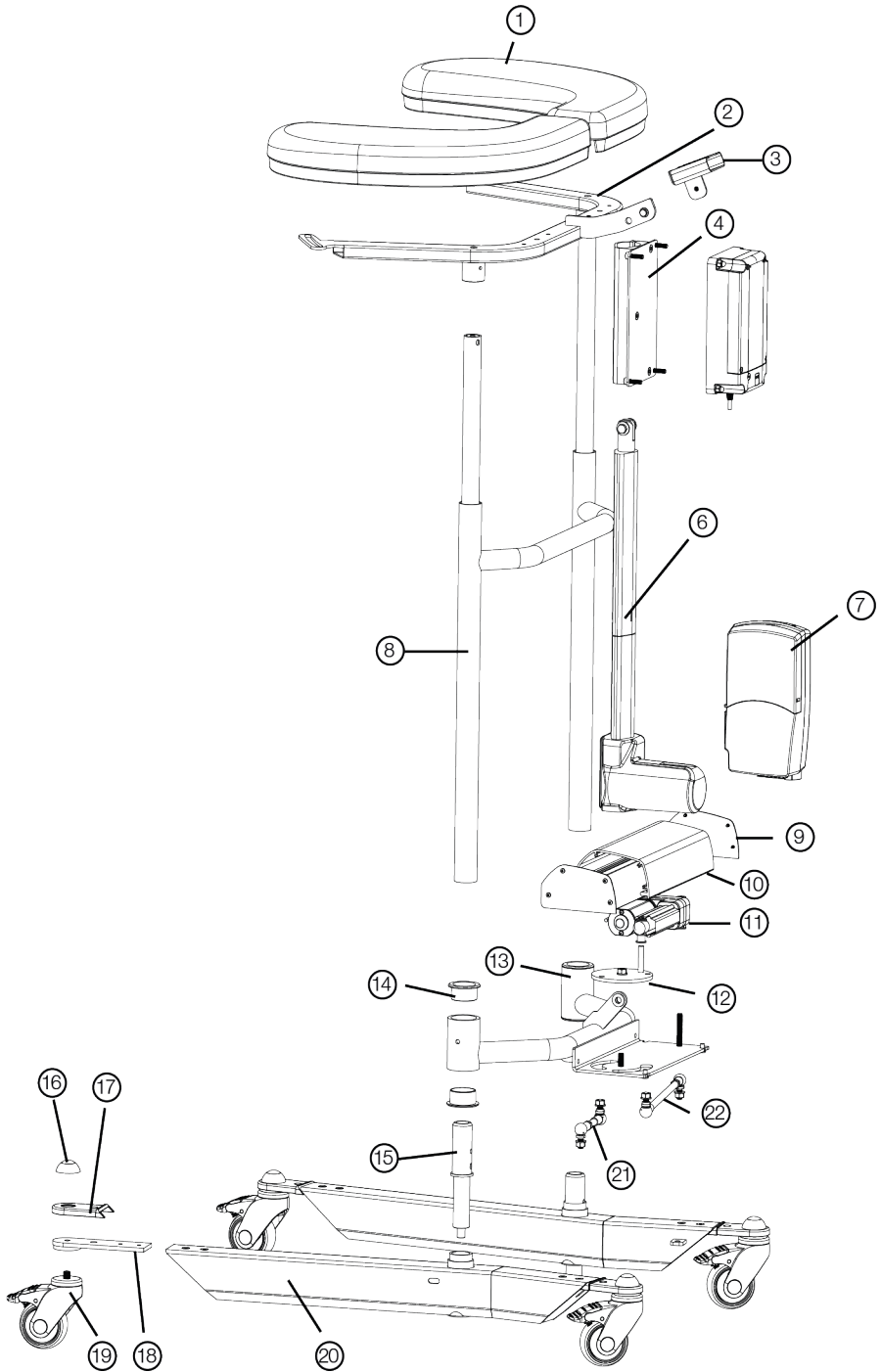
## 21. Expected lifetime:

The life expectancy of the product is 8 years.

## 22. Parts List

1	57-306 57-305	Armrestpad left Armrestpad right	12	58-364	Circular plate
2	58-365	Frame/Armrest	13	58-352	Cross section
3	57-041-3	Holder, Hand control (TH7)	14	58-302	Bearing (x2) Cross section
4	57-369 57-370	Clamp/Battery (alu) Plate Clamp/Battery (alu)	15	58-353	Turn axle, beam
5	58-376	Battery, TIMOTION	16	57-776 57-775	Cover, Wheel Washer, wheel
6	58-377	Actuator, TIMOTION	17	58-360	Cover, Wheelplate (PA)
7	58-373 58-373-1	Controlbox 220V, TIMOTION Controlbox 110V, TIMOTION	18	58-354	Wheel mount beam
8	58-366 58-335	Vertical frame Telescopic Innertube incl. Bronzebearing	19	57-051 57-319 57-049	Wheel 75mm Wheel 100mm Wheel 125mm
9	58-358	Sideplate (2x) alu	20	58-379-L 58-379-R	Bottom beam, left Bottom beam, right
10	58-356	Aluprofile, widening	21		Linking arm, right
11	58-378	Actuator, widening (TIMOTION)	22		Linking arm, left

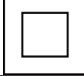
## 23. Accessories

<b>Art. Nr.</b>	<b>Description</b>
56-339-2-0	Manual brake, 2 handles
56-339-2-1	Manual brake, 1 handle
56-333-DB	Standing plate, Short
56-383-DB	Standing plate, Long
56-334	Dripstand, Chromed
56-335	Holder for Dripstand
56-336	Oxygen tubeholder
56-337	Side support
56-338	Dripstand holder, Complete
56-387	Wheel w brake 75 mm
56-358	Wheel w brake 100 mm
56-361	Wheel w brake 125 mm
57-361-L	Armrestpad PUR left
57-361-R	Armrestpad PUR right
58-374-2	Hand Control (TH7) TiMOTION
56-062	Hand Control PLUS, DB (TH10) TiMOTION
58-326 (-1, -2, -3)	Coiled Cable/EUR (-UK, -US, -AUS) TiMOTION
56-388-KIT	Kit/Gait training for Belt Multi
56-388	Gait training kit

## 24. Table of Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code
	Type BF, according to the degree of protection against electric shock		The device is intended for indoor use
	WEEE Symbol May not be discarded in domestic waste		Class II Equipment
	Duty Cycle: 2 min in active (ON) mode. 18 min in rest (OFF) mode.		Wash at 75 degrees
	Do not iron		Do not dry clean
	Do not bleach		

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# Electromagnetic Compatibility (EMC)

## Electromagnetic emissions and test levels

The product is intended to be used in the stated environments with electromagnetic levels as specified below. The caregiver and/or user of the product assure that the product is used in such an environment.

Emission test	Standards	Compliance
RF emissions	CISPR 11	Group 1
RF emissions	CISPR 11	Class B
Harmonic current emissions	IEC 61000-3-2	Class A
Voltage fluctuations and Flicker emissions	IEC 61000-3-3	Complies

## Electromagnetic Immunity

Acceptance criteria for the EMC pass criteria is unintentional movement above  $\pm 10\text{mm}$  is not allowed. The product is intended to be used in the stated environments with electromagnetic levels as specified below. The caregiver and/or user of the product assure that the product is used in such an environment.

Acceptance criteria for the EMC pass criteria is unintentional movement above  $\pm 10\text{mm}$  is not allowed. The product is intended to be used in the stated environments with electromagnetic levels as specified below. The caregiver and/or user of the product assure that the product is used in such an environment.

Radiated Fields in close proximity, Immunity test and compliance. Standard: IEC 61000-4-39			
Dwell time: 3 sec			
Test levels (A/m)	Modulation Pulse modulation	Mod. Frequency (kHz)	Test Frequency
8	CW	CW	30 kHz
65	50% Duty cycle	2.1	134.2 kHz
7.5	50% Duty cycle	50	13.56 MHz

Immunity test	Basic EMC standard or test method	IEC 60601-1-2 Edition 4 Test levels and compliance	
		Professional healthcare facility environment†	Home healthcare environment
Surge	IEC 61000-4-5	± 0.5 kV, ± 1 kV	
Voltage dips	IEC 61000-4-11	0% U <sub>T</sub> ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	
		0% U <sub>T</sub> ; 1 cycle and 70% U <sub>T</sub> ; 25/30 cycles. Single phase: at 0°	
Voltage Interruptions	IEC 61000-4-11	0% U <sub>T</sub> ; 250 / 300 cycle	
Conducted disturbances induced by RF fields	IEC 61000-4-6	6V in ISM bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	6V in ISM and amateur bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz
		Dwell time: 3 sec Frequency step size: 1%	
Radiated RF Electro-magnetic fields	IEC 61000-4-3	3 V/m 80 MHz – 2.7 GHz 80% AM at 1 kHz	10 V/m 80 MHz – 2.7 GHz 80% AM at 1 kHz
		Dwell time: 3 sec Frequency step size: 1%	
Electrical fast transients / burst	IEC 61000-4-4	± 2 kV 100 kHz repetition frequency	
RATED Power frequency magnetic field	IEC 61000-4-8	30 A/m 50 Hz or 60Hz	
Electrostatic Discharge (ESD)	IEC 61000-4-2	Contact: ± 8 kV Air: ± 2 kV, ±4 kV, ± 8 kV, ± 15 kV Number of discharges: 10 for each polarity	

Radiated Radio – Frequency, Proximity fields from wireless communication equipment & compliance. (Dwell time: 3 sec.)  
 Standard: IEC 61000-4-3

	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Immunity test levels (V/m)
Spot Frequencies	385	380 to 390	TETRA 400	Pulse modulation 18 Hz	27
	450	430 to 470	FMRS 460, FRS 460	FM ± 5 kHz Deviation 1 kHz sine	28
	710	704 to 787	LTE Band 13, 17	Pulse modulation 217 Hz	9
	745				
	780				
	810	800 to 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	28
	870				
	930				
	1720	1700 to 1990	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 217 Hz	28
	1845				
	1970				
	2450	2400 to 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	28
	5240	5100 to 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	9
	5500				
5785					
Other Identified frequencies	433	-	-	Pulse modulation 2 Hz	3
	-	860-960	-	Pulse modulation 2 Hz	54
	2450	-	-	Pulse modulation 2 Hz	54

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

Type BF Applied Parts

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.