Moving Health Forward

**EN** [User Manual](#)

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
# LEJRELET

## Pad High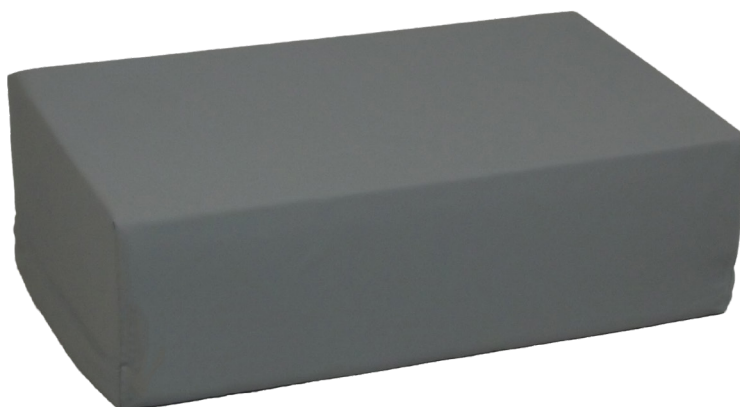

# EN

## Application


LEJRELET Pad High is a rectangular cushion made from a mixture of polyurethane foam and viscoelastic foam. It is used for various positioning situations and during lower hygiene procedures for the bedridden patient.

LEJRELET Pad High has a large support surface which in combination with viscoelastic foam offers good pressure relief and comfort for the bedridden patient.


Pad High can be used alone or in combination with other LEJRELET products.

Examples for use of LEJRELET Pad High:


- Support of leg in lateral position
- Support of leg during lower hygiene procedures
- Support of arm in lateral position
- Support of legs and pressure relief of heels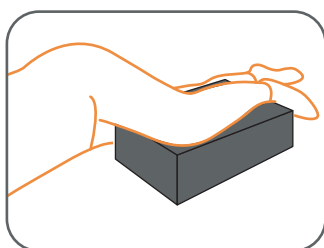


*Support of leg in lateral position*

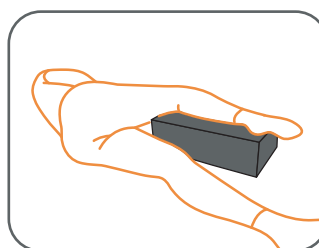*Support of leg during lower hygiene situations*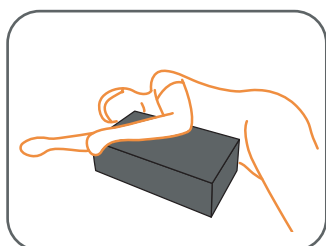

*Support of arm in lateral position*

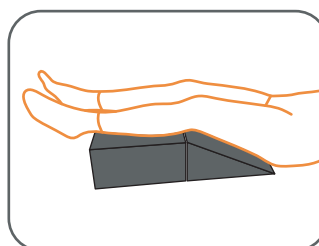*Support of legs and pressure relief of heels*

## Technical Information

### Filling:

LEJRELET Pad High is pressure relieving and made with a core of poly-urethane foam covered with 2 cm of flame retardant viscoelastic foam on the top.

This combination means, that the cushions are firm and supportive, yet they are still flexible and provide effective pressure relief and comfort.

### Fabric:

- Waterproof
- Water vapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

## Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

**Size:** H 15 x W 30 x L 50

**Item no.:** 5000003/5000021

**Weight:** 800 g

## Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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**This Page is Left Intentionally Blank**

**This Page is Left Intentionally Blank**

Moving Health Forward

**PRESSURE ULCER PREVENTION**  
**SAFE MOVING & HANDLING**  
**SPECIALIST THERAPIES**  
**BATHROOM SAFETY**  
**RENTAL & SERVICE SOLUTIONS**

Direct Healthcare Group Ltd  
Withey Court, Western Industrial Estate  
Caerphilly, United Kingdom  
CF83 1BF

T: +44 (0) 800 043 0881

F: +44 (0) 845 459 9832

E: [info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)

**EC** **REP**

**EU Authorised Representative, Importer, and  
Distributor:**

Direct Healthcare Group Sverige AB,  
Torshamnsgatan 35, SE-164 40, Kista  
Sweden  
T: +46 (0) 8 557 200

**Australian Sponsor:**

Direct Healthcare Group Pty Ltd.,  
68 Howe Street, Osborne Park,  
Western Australia 6017  
T: +61 (0) 423 852 810

Issue 4

Date: November 2024

[DHG-HEALTHCARE.COM](http://DHG-HEALTHCARE.COM)