SafeHandlingSheet

System **RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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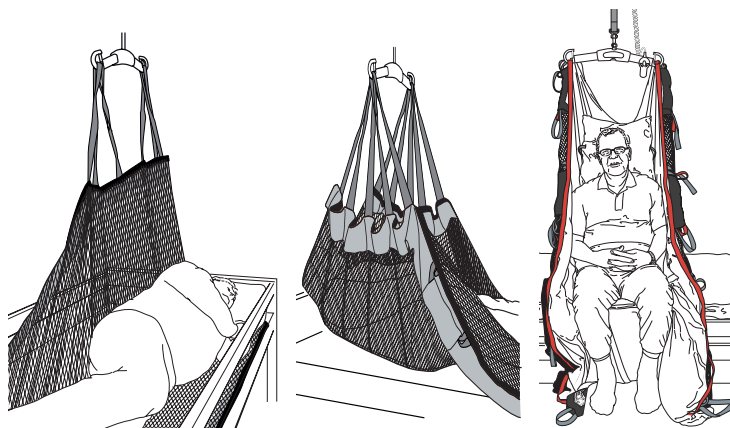
REF

47522007


47542007

47592008

Max. patient weight: 300 kg/660 lbs

Table of contents

English	3
Svenska	8
Norsk	13
Suomi	18
Deutsch	23
Nederlands	28
Français	33
Italiano	38
Symbols	43

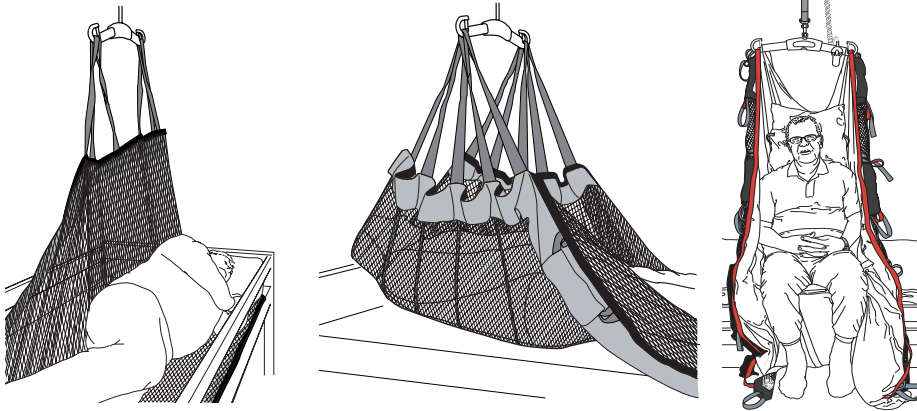SafeHandlingSheet is a sheet that can be used for repositioning in bed with both stationary and mobile lifts. It features sixteen suspension points, eight on each side of the patient. SafeHandlingSheet is designed to meet the need for repositioning patients in bed.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The choice of model and material depends on the patient's needs and the transfer situation.

Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

The caregiver must be able to read and understand the manual/Instructions for use of the product.

Caregiver: Never use the device if you have not had a proper training on how to use the device including the accessories.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully.

First, apply the sling's back loops to the slingbar, and then apply the leg support loops.

Use a slingbar that has been tested.

The lowest allowable safe working load always determines the safe working load of the assembled system.

Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

Description of SafeHandlingSheet:

SHSheet L Polyester net/WendyLett4Way

SafeHandlingSheet has lift straps along both sides.

To simplify handling for the caregiver, the lift straps are alternately grey and black.

The lift straps have three loop alternatives with different colours: grey, red and black.

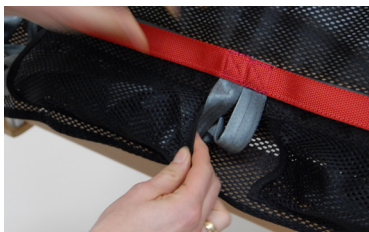
SafeHandlingSheet is marked with a decal to show which side should be placed upwards on the bed. "This side up".

The decal also marks the middle of the sheet.

SHSheet L Polyester net


When repositioning is completed, bed the straps in under the mattress or place them in the sidepockets.

Tip!

- Use with plain sheets on top of the SafeHandlingSheet.
- Place the user in the center of the SafeHandlingSheet.
- The extra material available with the SafeHandlingSheet allows the user to be repositioned several times before you need to reposition the user on the SafeHandlingSheet.
- Recommended for use with traversing (x-y axis) ceiling lift systems.
- If possible, consider moving the bed in the first place instead of shoot / draw the user when he / she lifted from the surface.

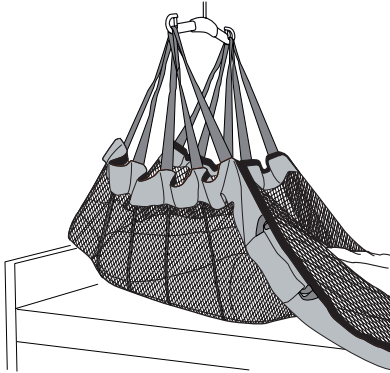
Use of SafeHandlingSheet:

Higher up in bed:

The five uppermost straps on either side (or as many straps as are required to cover the friction points) are hooked to the slingbar.

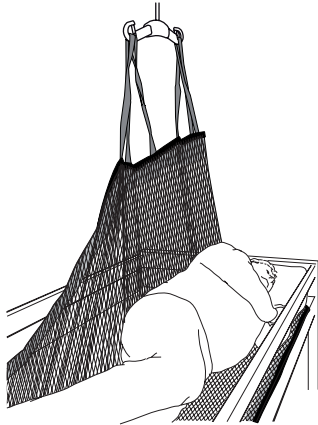Start raising the lift and move the patient higher up in bed when weight is relieved from the friction points.

Turning in bed:

If the user is to be positioned in the center of the bed after turning, it is recommended to first reposition the user (see: Higher Up in Bed) laterally to one side of the bed and then roll the user towards the center of the bed.

To perform the roll, connect the four center-most lift straps on the side of the SafeHandlingSheet (side opposite the direction of the intended roll) to the sling bar. If bed rails are not used, a caregiver should stand on either side of the bed to ensure the safety of the user. Raise the lift to gently turn the patient towards the center of the bed.

Change of SafeHandlingSheet:

When replacing an old SafeHandlingSheet with a new one, start with a lift as previously described. When the user is lifted clear of the bed, place the new SafeHandlingSheet centered on the bed. Lower the user back down to the bed and disconnect the old SafeHandlingSheet from the sling bar.

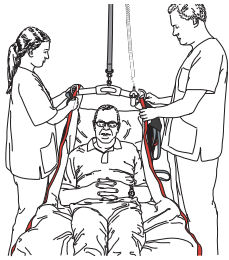
Connect the new SafeHandlingSheet to the sling bar and roll the user onto their side (see: Turning in Bed) and roll / tuck the old SafeHandlingSheet behind the user, then roll the user back down onto the bed. Do another turn with the new SafeHandlingSheet in the opposite direction and pull away the old SafeHandlingSheet from behind the user, then roll the user back down onto the bed.

Sit-to-stand – from a seated position at the bedside

Note! For this sit-to-stand procedure, the patient must be able to sit at the bedside.


Lower the bed, so that the patient can place his/her feet firmly on the floor when seated at the bedside.

If the bed is too high, use a foot stool.


Prepare the patient for raising.

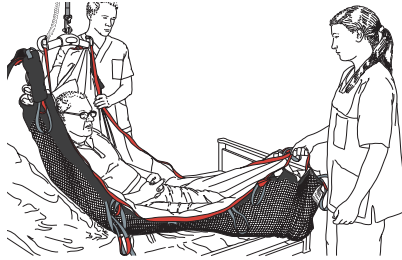Raise the backrest on the bed and place the uppermost handle straps on the slingbar.

One caregiver should grasp the handles at the foot end.

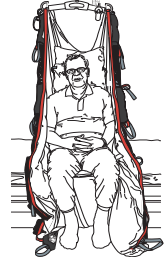Start by raising the lift. The caregiver lifts the patient's legs over the edge of the bed and places the feet on the floor. Make sure the patient can sit securely at the bedside. Knees and hips should be at the same level. Place a suitable standing frame or walker in front of the patient to aid standing.

Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions.

The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on www.directhealthcaregroup.com

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer.

The periodic inspection instructions for this product are found on www.directhealthcaregroup.com

Product	Art No.	Measures	Material	Max. patient weight
SHSheet L poly net	47522007	240 x 115cm	Polyester net	300kg / 660 lbs.
SHSheet L Poly net/ WendyLett4Way	47542007	240 x 115cm	Polyester net + polycotton	300kg / 660 lbs.
SHSheet Disposable	47592008	220 x 135cm	Non-Woven polypropylene	300kg / 660 lbs.

Care of the product:

47522007

47542007

Read the product label.

Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

Care of the product:

47592008

Read the product label.

Do not wash the

Disposable SHSheet

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

How to report a serious incident

Any serious incident that has occurred in relation to the sling should be reported to the manufacturer and the MHRA or another competent authority of the country in which the user and/or patient is established.

UK

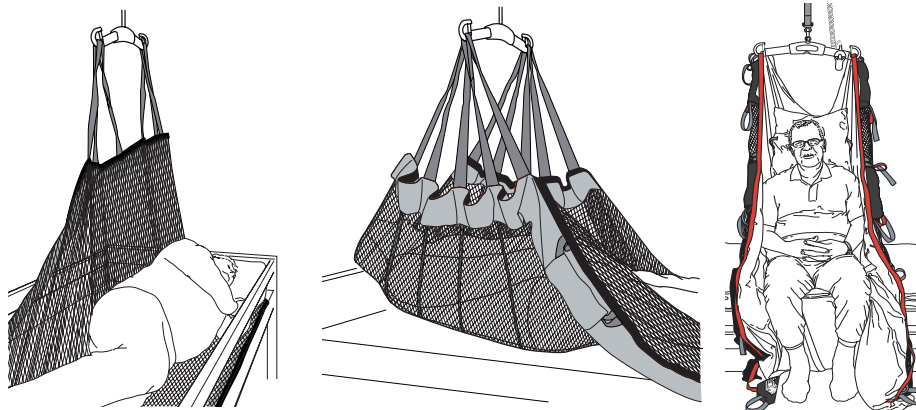
info@directhealthcaregroup.com

T: +44 (0) 800 043 0881

F: +44 (0) 845 459 9832

Other:

info.export@directhealthcaregroup.com

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T: +46 8-557 62 200

Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code
	Maximum patient weight		

Simple solutions for great results

SystemRoMedic[®] is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic[®] is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic[®] is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic[®] offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;

www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic[®] representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.